

Brian F. King, M.D.
Director of Neuroradiology

Douglas H. Brown, M.D.
Director of Musculoskeletal Imaging

Jeffrey M. Silverman, M.D.
Director of Body Imaging

Phillip Hahn, M.D.
Neuroradiology

HOW TO VIEW IMAGES AND REPORTS ON THE WEB

LAUNCH AND LOGIN

1. Open Internet Explorer - go to **www.Landmarkimaging.com**.
2. Click on **View Images and Reports**.
3. Always **answer "Yes" to security questions and click the ActiveX control bar** if it pops up. If you receive an error, choose the option to "Continue to this Website".
4. **Type your user name and password** given to you by the Landmark Imaging Marketing Coordinator:

User Name _____ Password _____

5. **Click Login** (or use Enter key).

If the login fails, check the Caps Lock key; usernames and passwords are case sensitive. Also check the pop-up bar to allow ActiveX controls, or add the Dynamic Imaging demo server as a Trusted Site in the Security settings panel of the Internet Options dialog.

FIND PATIENT RECORD

1. Click on the **binoculars in the upper left hand corner**.
2. **Enter your patient's last name** to display the patient's images and reports.

STUDY LIST

Use the **paging buttons on the toolbar above the tabs to navigate forward or backward through the Study List**.

- If the notepad icon is green, then the report has been attached to the study.

OPEN A STUDY

Click anywhere on the study's row in the Study List to launch Image Viewer and load a patient's exam.